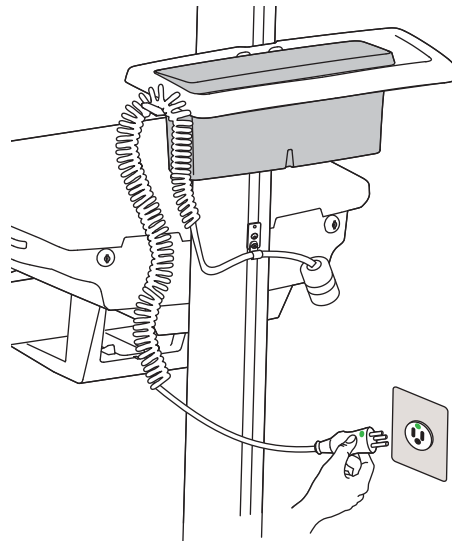
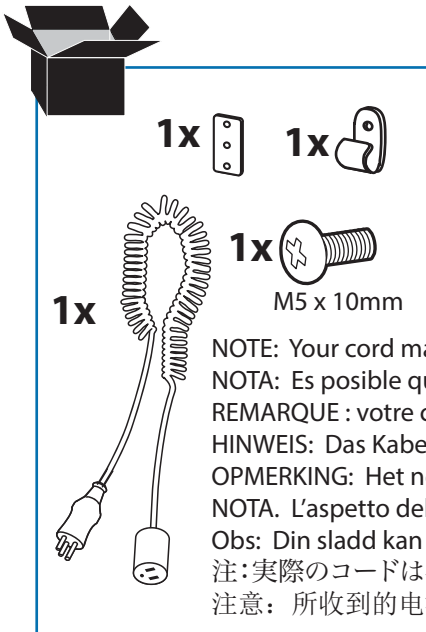
ergotron®

Coiled Cord for StyleView Non-Powered Cart

Hospital grade power outlet.
Enchufe eléctrico de calidad hospitalaria.
Prise de courant de qualité hospitalière.
Steckdose (Hospital Grade)
Stopcontact goedgekeurd voor ziekenhuizen.
Presa di corrente per strutture ospedaliere.
Eluttag av sjukhustyp.
病院仕様の電源出力
医用插座。

NOTE: Your cord may look different than the one illustrated.
NOTA: Es posible que su cable tenga un aspecto distinto al de la ilustración.
REMARQUE : votre câble peut être différent du câble illustré ici.
HINWEIS: Das Kabel unterscheidet sich ggf. von der Darstellung.
OPMERKING: Het netsnoer kan er anders uitzien dan in de illustratie.
NOTA. L'aspetto del cordone può essere diverso da quello illustrato.
Obs: Din sladd kan se annorlunda ut än den som visas.
注: 実際のコードはイラストと異なる場合があります。
注意: 所收到的电源线可能与示意图中的不同。

EN User Guide

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








www.ergotron.com | USA: 1-800-888-8458 | Europe: +31 (0)33-45 45 600 | China: 400-120-3051 | Japan: japansupport@ergotron.com

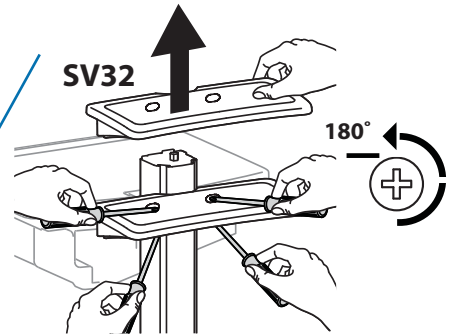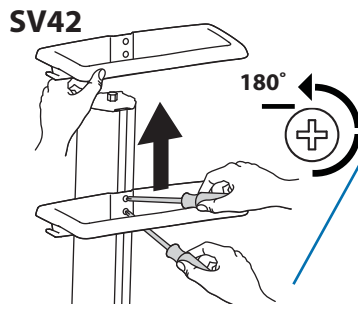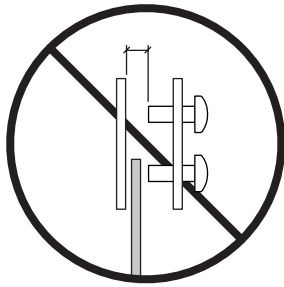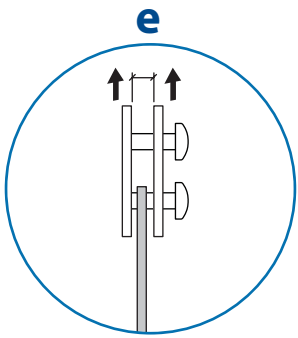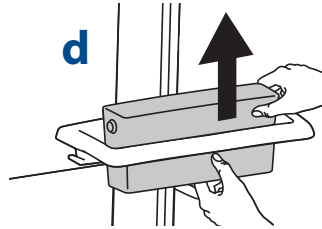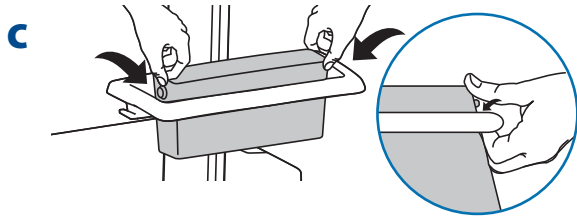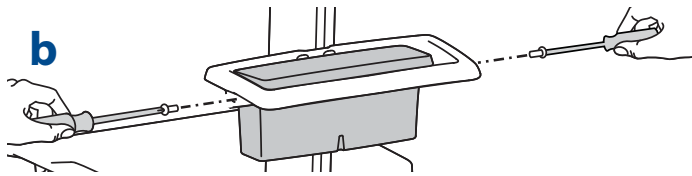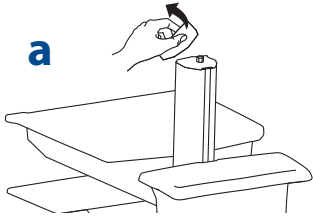1


2


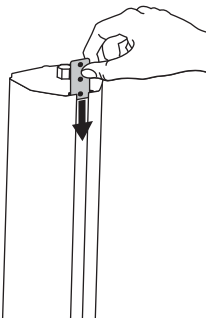
a 1x


b

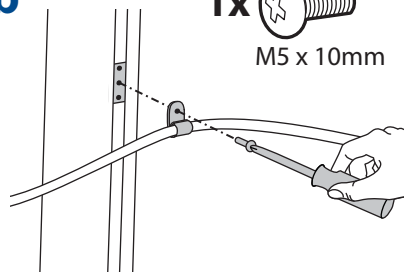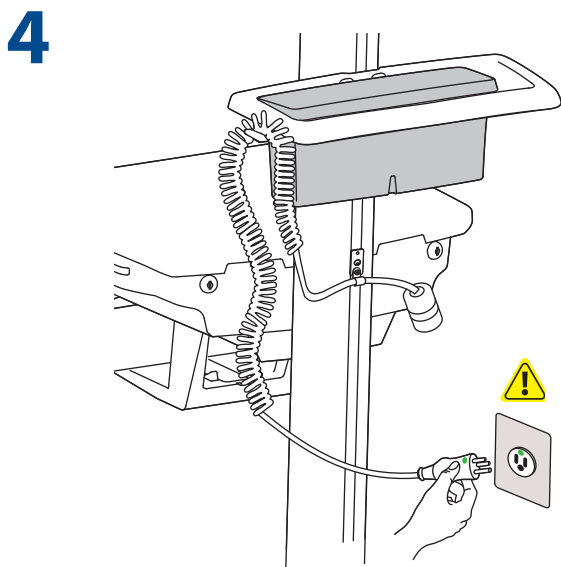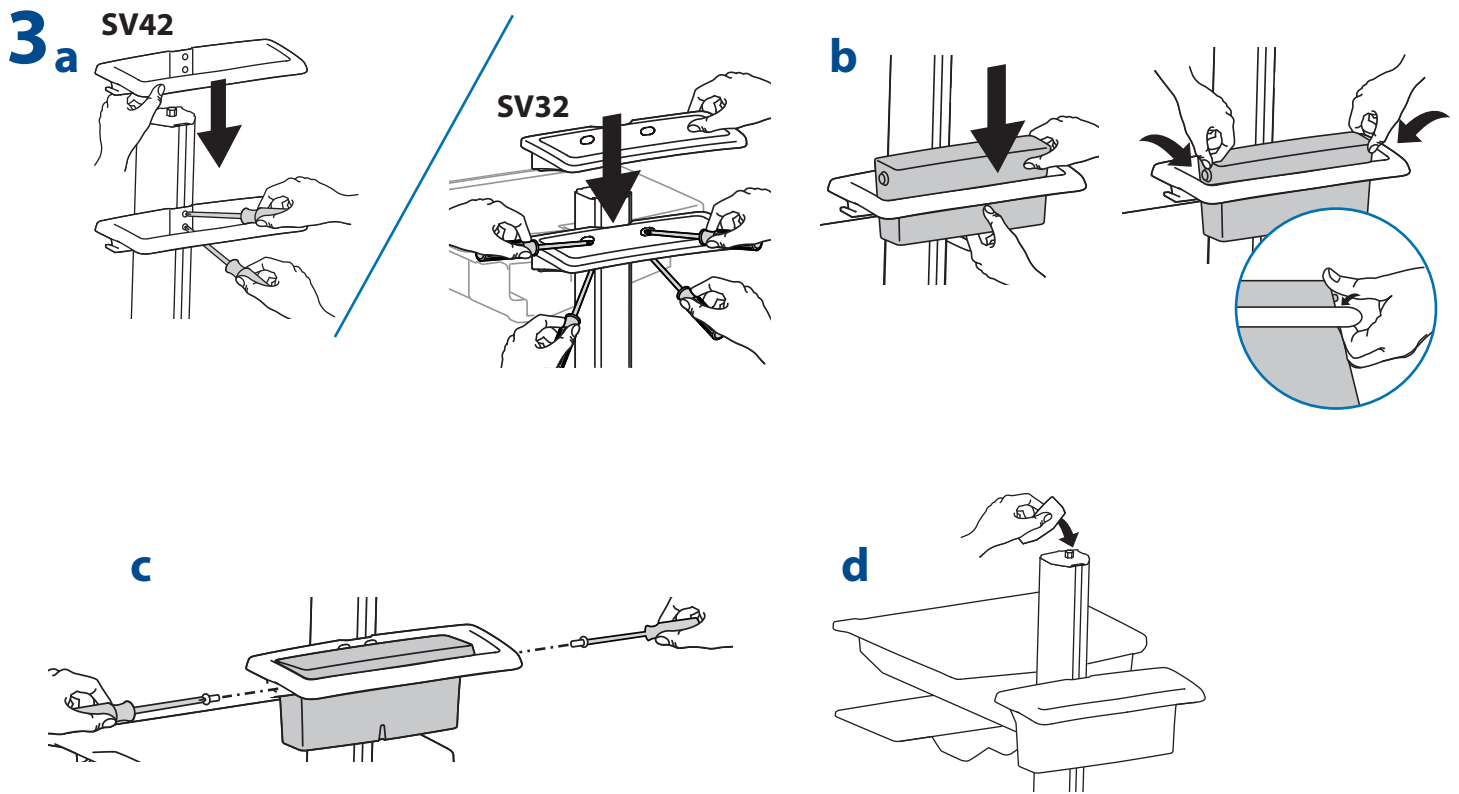1x 1x
M5 x 10mm

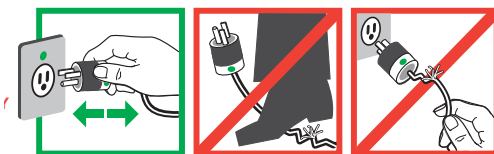Hospital grade power outlet.
 Enchufe eléctrico de calidad hospitalaria.
 Prise de courant de qualité hospitalière.
 Steckdose (Hospital Grade)
 Stopcontact goedgekeurd voor ziekenhuizen.
 Presa di corrente per strutture ospedaliere.
 Eluttag av sjukhustyp.
 病院仕様の電源出力
 医用插座。

⚠️ DANGER!

ELECTRICAL CORDS CAN BE HAZARDOUS
Misuse Can Result in FIRE or DEATH by ELECTRICAL SHOCK.
Please Read and Follow Product Manual Instructions.
THIS IS A POLARIZED CORD - HOSPITAL GRADE ONLY!
NOTE: Grounding Reliability Can Only Be Achieved When the Equipment Is Connected To An Equivalent Receptacle Marked "HOSPITAL ONLY" Or "HOSPITAL GRADE"

- Inspect Cord Thoroughly Before Each Use. DO NOT USE IF DAMAGED.
- Do Not Plug More Than SPECIFIED NUMBER OF WATTS Into Power System.
- Do Not Run Cord Through Doorways, Holes in Ceilings, Walls or Floors.
- FULLY INSERT Plug Into Outlet.
- Do Not Remove, Bend or Modify Any Metal Prongs or Pins of Cord.
- Do Not Use Excessive Force to Make Connections.
- Keep Away From Water. DO NOT USE WHEN WET.
- Keep Children Away From Cord.
- Do Not Plug Cord into An Extension Cord.
- AVOID OVERHEATING. Uncoil Cord and Do Not Cover It With Any Material.
- Do Not Drive, Drag or Place Objects Over Cord. Do Not Walk On Cord.
- GRASP PLUG TO REMOVE FROM Outlet. Do Not Unplug By Pulling On Cord.

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© 2014 Ergotron, Inc. All rights reserved.

While Ergotron, Inc. makes every effort to provide accurate and complete information on the installation and use of its products, it will not be held liable for any editorial errors or omissions (including those made in the process of translation from English to another language), or for incidental, special or consequential damages of any nature resulting from furnishing this instruction and performance of equipment in connection with this instruction. Ergotron, Inc. reserves the right to make changes in the product design and/or product documentation without notification to its users. For the most current product information, or to know if this document is available in languages other than those herein, please contact Ergotron. No part of this publication may be reproduced, stored in a retrieval system, or transmitted in any form or by any means, electronic, mechanical, photocopying, recording or otherwise without the prior written consent of Ergotron, Inc., 1181 Trapp Road, Eagan, Minnesota, 55121, USA Patents Pending and Patented U.S. & Foreign. Ergotron is a registered trademark of Ergotron, Inc.